

Instructions for use

COMBILOC, clear **Ø 120 mm**



Double-coated plates

Please store in a cool and dry area.

Application area

Comfortable periodontal and bite splints

Material

Soft/hard abrasion protected joining plate. The hard side can be bonded with acrylic. After having applied the acrylic, we recommend to put the splint aside for about 2 hours. The model should be isolated with silicon spray or water e.g. The flexible and soft side is very comfortable for the patient when wearing it.

Characteristics

Bio-compatible material made of EVA and SBS. Not water-soluble, recyclable, odourless, no danger for the underground water. Can be disposed in the household waste. Low emission, high energy waste when burning (waste incineration).

Colour

Transparent

Modelling

Before modelling, the model must be prepared: trimmed, sharp edges have to be removed, it has to be blocked out and insulated.

For the elaboration, we recommend our processing set for thermoforming sheets (item-no. 581-004-030). We recommend for rough elaboration our HSS-pointed bur (item-no. 581-004-031), for fine elaboration (item-no. 581-004-032), especially for the interdental area our HSS-spiral drill (item-no. 581-004-033). For smoothing and polishing, we recommend our polishing disc brown (item-no. 581-004-042) and our polisher dark grey (item-no. 581-004-043).

Assortment:

581-012-092	thickness 1.8 mm	content 10 pcs.
581-012-094	thickness 3.0 mm	content 10 pcs.
581-012-173	thickness 1.8 mm	content 50 pcs.
581-012-175	thickness 3.0 mm	content 50 pcs.