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**Intended use**

Dental splinting material long-term

**Extended purpose**

Soft/hard thermoforming foil made of polyethylene terephthalate and ethyl vinyl acetate for the production of functional splints such as occlusion or bruxism splints.

**Indications for use**

Due to its properties, the material is suitable for the production of bite and bruxism splints, as well as for the treatment of myoarthropathies (muscle/jaw joint disorders) and resulting parafunctions such as clenching, grinding etc.

**Kontraindication**

In case of allergies to any of the ingredients do not use **COMBILOC PLUS**. Not desired side effects of this medical device are not to be expected in case of proper processing and application. However, immune reactions (e. g. allergies) or irritations principally cannot be excluded. Should you get to know non-desired side effects – also in doubtful cases – we appreciate in any case to be informed with a detailed description of the attendant circumstances and symptoms. We will follow each hint.

**Target patient group**

Patients who need dental care.

**Intended users**

Trained staff in dental field

**Characteristics**

Double-layer Foil with soft/hard material combination, made of polyethylene terephthalate and ethyl vinyl acetate, with insulating foil
Ø 120 mm / 125 mm
□ 125 mm x 125 mm

**Material thickness**

2.0 / 3.0 / 4.0 mm

**Processing**

All **COMBILOC PLUS** materials are delivered with an insulating foil. This thin foil (0,07 mm) is thermoformed together with the

**COMBILOC PLUS** on the model side and guarantees a decisively improved transparency of the splint. Thus an additional insulation of the model surface is no longer necessary. The heating-up of the **COMBILOC PLUS** foils is done from the hard side (the foil is only suitable for pressure forming units).

The heating and cooling times can be taken from the table.

**Cleaning and disinfection**

The adhesive surfaces must not come into contact with PMMA or other alcoholic cleaning agents. Alcohol-based disinfection may also lead to a delamination of the two foils. For cleaning, the use of MaxiFresh™ cleaning tablets is recommended. Please be aware of the MaxiFresh™ instruction for use.

**Note**

While working out on the material, it is recommended to use an exhaust system and protective glass or goggles as well as a face mask.

**Serious incidents**

All serious incidents occurring in relation to the product shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/ or patient is settled.

**Storage conditions**

At room temperature, i. e. 18–28 °C.

Protect against humidity, always close foil bag when packs are in use.

**Lot number / Expiry date**

The lot number and the expiry date are indicated on each packaging. In case of claims, please always indicate the lot number of the product. Do not use the product after the expiry date.

**Disposal**

Disposal of contents/the packaging in accordance with local/regional/national/international regulations

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This product was developed for dental use and has to be used according to the instructions. In the case of damages resulting from ulterior or improper use, the manufacturer assumes no liability. Furthermore the user is bound to check the product according to suitability and possible use in self dependence

before using it subject to its purpose, especially in case the purpose is not mentioned in the instructions.

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