Ámbitos de aplicación

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Field of application

FOLILEN is especially suitable for the production of cast form parts for the c & b technique. It serves as a stable basis and determination of the minimum strength for wax set-up in the crown and bridge technique. Furthermore, FOLILEN can be used as a space maintainer for bleaching resp. medicament supports. FOLILEN does not bond with acrylate and burns without residues.

Characteristics

Soft-elastic thermoforming foil made of polyethylene
 Ø 120 mm / 125 mm
 □ 125 mm x 125 mm

Material thickness

0.45 / 0.65 / 0.8 / 1.0 mm

Color

nature

Processing

Regarding the heating times please refer to the scale below:

FOLILEN	 Drufoformat/Drufosmart
0.45 mm	1:10
0.65 mm	1:15
0.80 mm	1:25
1.00 mm	1:40

Before the thermo-forming process the plaster models should be watered and insulated with an insulating agent on alginate basis. The blank is plastified in a suitable thermo-forming unit. By means of vacuum or pressure the plastified plate is formed on the object. The cooling down phase under pressure / vacuum should at least correspond to the respective heating time. FOLILEN can be reworked with scissors.

In order to receive a reservoir for the bleaching material when using the bleaching splint a space maintainer has to be applied onto the model.

For the production of space maintainers for bleaching resp. medicament supports the blank is removed from the model after the cooling down phase which is effected under pressure / vacuum and is cut at the areas which should be bleached. The blank slightly overlaps the incisal edges of the anterior teeth and end approx. 1 mm in front of the gingival margin. The space maintainer is placed back onto the model and fixed. The FOLIFLEX BLEACH blank is thermo-formed on the model and after a cooling down phase of at least 3:00 min. the blank can be removed.

Lot number / Expiry date

The lot number and the expiry date are indicated on each FOLILEN packaging. In case of claims, please always indicate the lot number of the product. Do not use the product after the expiry date.

Storage conditions

At room temperature, i.e. 18 °C–28 °C.

Durability: see date of expiry.

FOLILEN is not to be used when expiry date has passed.

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This product was developed for dental use and has to be applied according to the instructions. In the case of damages resulting from ulterior or improper use, the manufacturer assumes no liability. Furthermore the user is bound to check the product according to suitability

and possible use in self dependence before using it subject to its purpose, especially in case the purpose is not mentioned in the instructions.

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