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Intended use
Dental splinting material long-term

Extended purpose
Soft thermoforming foil for the production of mouthguards, positioners and various occlusion splints.

Indications for use
Due to its properties, the material is suitable for the production of bite and bruxism splints, as well as for the treatment of myoarthropathies (muscle/jaw joint disorders) and resulting parafunctions such as clenching, grinding etc. It can also be used as a positioner for slight tooth movements.

Cleaning
For cleaning, the use of MaxiFresh™ cleaning tablets is recommended.
Please be aware of the MaxiFresh™ instruction for use.

Contraindication
In case of allergies to any of the ingredients do not use FOLIFLEX. Not desired side effects of this medical device are not to be expected in case of proper processing and application. However, immune reactions (e.g. allergies) or irritations principally cannot be excluded. Should you get to know non-desired side effects – also in doubtful cases – we appreciate in any case to be informed with a detailed description of the attendant circumstances and symptoms. We will follow each hint.

Storage conditions
At room temperature, i.e. 18–28 °C.

Target patient group
Patients who need dental care.

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cooling time under pressure / vacuum is 3–8:30 minutes. After the cooling time FOLIFLEX can be cut by means of scissors or a scalpel. FOLIFLEX is suitable for "additive thermoforming". Through process of lamination various foils can be connected **only with pre-forming unit under pressure. Heating and cooling times can be taken from the table. Before thermoforming plaster models should be watered and insulated.**

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Lot number / Expiry date
The lot number and the expiry date are indicated on each packaging. In case of claims, please always indicate the lot number of the product. Do not use the product after the expiry date.

Disposal
Disposal of contents/the packaging in accordance with local/ regional/national/international regulations.

Characteristics
Flexible, soft-elastic thermoforming foil made of ethyl-vinyl-acetate (EVA)
 Ø 120 mm / 125 mm
 125 mm x 125 mm

Material thickness
1.0 / 1.5 / 2.0 / 3.0 / 4.0 / 5.0 mm

Color
nature-transparent

Processing
For using FOLIFLEX foils a suitable (dental) thermoforming device (pre-forming unit under pressure or vacuum) is required. When using it, always follow the manufacturer's instructions. For plasticization the blank will be placed into the plate reception and with the help of the radiant heater brought to the necessary temperature. The times vary according to the unit type and the surrounding temperature. After end of the heating time the material hangs down → complete the thermoforming process. Depending on the material thickness the

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This product was developed for dental use and has to be used according to the instructions. In the case of damages resulting from superior or improper use, the manufacturer assumes no liability. Furthermore the user is bound to check the product according to suitability and possible use in self dependence before using it subject to its purpose, especially in case the purpose is not mentioned in the instructions.

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